

How to Connect the AiCure Provisioned Device to WiFi

Q. How can I (or my participant) connect the AiCure Provisioned Device to the WiFi?

A. Please follow the steps below to connect the device to WiFi:

1. Power ON the AiCure provisioned device
 - Connect to a charger if the battery is LOW
2. On the main home screen, tap on **SETTINGS**
3. Tap on **WiFi**
4. Choose the appropriate WiFi network (Fast and secure connection preferred)
5. Look for "Connected" under the network as well as the WiFi symbol at the top left of the notifications bar without an exclamation point

Please Note

If the device shows the WiFi symbol with an exclamation mark, it is NOT connected to WiFi and may require a password.

